

# Zyto Light SPEC MYC Dual Color Break Apart Probe

**REF** Z-2090-50

 $\sum$ 

5 (0.05 ml)

**REF** Z-2090-200

 $\sqrt{\Sigma}$ 

20 (0.2 ml)

For the qualitative detection of translocations involving the human MYC gene at 8q24.21 by fluorescence *in situ* hybridization (FISH)





In vitro diagnostic medical device according to EU directive 98/79/EC

### 1. Intended use

The Zyto Light SPEC MYC Dual Color Break Apart Probe (PL49) is intended to be used for the qualitative detection of translocations involving the human MYC gene at 8q24.21 in cytologic or formalin-fixed, paraffinembedded specimens such as lymphoma tissue by fluorescence *in situ* hybridization (FISH). The probe is intended to be used in combination with the Zyto Light FISH Implementation Kits (Prod. No. Z-2028-5/-20, or Z-2099-20).

Interpretation of the results must be made within the context of the patient's clinical history with respect to further clinical and pathologic data of the patient by a qualified pathologist.

### 2. Clinical relevance

The MYC proto-oncogene (MYC proto-oncogene, bHLH transcription factor, a.k.a. CMYC) encodes a transcription factor essential for cell growth and proliferation and is broadly implicated in tumorigenesis. Translocations involving the MYC gene are considered to be cytogenetic hallmarks for Burkitt Lymphoma but are also found in other types of lymphomas. The most frequent translocation involving the MYC gene region is t(8;14)(q24.21;q32.3) juxtaposing the MYC gene in 8q24.21 next to the IgH (immunoglobulin heavy chain) locus in 14q32.33. Further translocations affecting the MYC gene are t(8;22)(q24.21;q11.2) and t(2;8)(p11.2;q24.21), both of which involve one of the two immunoglobulin light chain loci. All three translocations bring the MYC gene under the control of a regulatory element from one of the immunoglobulin loci resulting in constitutive overexpression of MYC.

# Test principle

The fluorescence *in situ* hybridization (FISH) technique allows for the detection and visualization of specific nucleic acid sequences in cell preparations. Fluorescently-labeled DNA fragments, so called FISH probes, and their complementary target DNA strands in the preparations are codenatured and subsequently allowed to anneal during hybridization. Afterwards, unspecific and unbound probe fragments are removed by stringency washing steps.


After counterstaining the DNA with DAPI, hybridized probe fragments are visualized using a fluorescence microscope equipped with excitation and emission filters specific for the fluorochromes with which the FISH probe fragments have been directly labeled.

## 4. Reagents provided

The ZytoLight SPEC MYC Dual Color Break Apart Probe is composed of:

- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides ( $\sim 10.0$  ng/ $\mu$ l), which target sequences mapping in 8q24.21\* (chr8:130,373,051-130,930,673) distal to the MYC breakpoint region (see Fig. 1).
- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/μl), which target sequences mapping in 8q24.21\* (chr8:127,888,765-128,363,281) proximal to the MYC breakpoint region (see Fig. 1).
- Formamide based hybridization buffer

\*according to Human Genome Assembly GRCh37/hg19

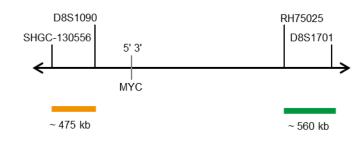

The Zyto Light SPEC MYC Dual Color Break Apart Probe is available in two sizes:

- Z-2090-50: 0.05 ml (5 reactions of 10  $\mu$ l each)
- Z-2090-200: 0.2 ml (20 reactions of  $10 \mu l$  each)

# 5. Materials required but not provided

- Positive and negative control specimens
- Hybridizer or hot plate
- Hybridizer or humidity chamber in hybridization oven
- Time
- Staining jars or baths
- Calibrated thermometer
- Adjustable pipettes (10 μl, 25 μl)
- Ethanol or reagent alcohol
- Deionized or distilled water
- Coverslips (22 mm x 22 mm, 24 mm x 60 mm)
- Rubber cement, e.g., <u>Fixogum Rubber Cement</u> (Prod. No. E-4005-50/-125) or similar
- Adequately maintained fluorescence microscope (400-1000x)
- Immersion oil approved for fluorescence microscopy
- Appropriate filter sets

# Cytology Specimens

- <u>Zyto Light</u> FISH-Cytology Implementation Kit (Prod. No. Z-2099-20)
- Microscope slides, uncoated
- Water bath (70°C)
- 37% formaldehyde, acid-free, or 10% formalin, neutrally buffered
- 2x Saline-Sodium Citrate (SSC), e.g., from 20x SSC Solution (Prod. No. WB-0003-50)

# **FFPE Specimens**

- <u>Zyto Light FISH-Tissue Implementation Kit</u> (Prod. No. Z-2028-5/-20)
- Microscope slides, positively charged
- Water bath (37°C, 98°C)
- Xvlene

## 6. Storage and handling

Store at 2-8°C in an upright position protected from light.

Use protected from light. Return to storage conditions immediately after use. Do not use reagents beyond expiry date indicated on the label. The product is stable until expiry date indicated on the label when handled accordingly.

#### 7. Warnings and precautions

- Read the instruction for use prior to use!
- Do not use the reagents after the expiry date has been reached!
- This product contains substances (in low concentrations and volumes) that are harmful to health and potentially infectious. Avoid any direct contact with the reagents. Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- If reagents come into contact with skin, rinse skin immediately with copious quantities of water!
- A material safety data sheet is available on request for the professional user.
- Do not reuse reagents.
- Avoid cross-contamination of samples as this may lead to erroneous results.
- The probe should not be exposed to light, especially strong light, for a longer period of time, i.e., all steps should be accomplished, where possible, in the dark and/or using lightproof containers!

# Hazard and precautionary statements:

The hazard determining component is Formamide.



# Danger

<b>V</b>	
H351	Suspected of causing cancer.
H360FD	May damage fertility. May damage the unborn child.
H373	May cause damage to organs through prolonged or repeated exposure.
P201	Obtain special instructions before use.
P202	Do not handle until all safety precautions have been read and understood.
P260	Do not breathe dust/fume/gas/mist/vapours/spray.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P308+P313	IF exposed or concerned: Get medical advice/attention.
P405	Store locked up.

# 8. Limitations

- For in vitro diagnostic use.
- For professional use only.
- The clinical interpretation of any positive staining, or its absence, must be done within the context of clinical history, morphology, other histopathological criteria as well as other diagnostic tests. It is the responsibility of a qualified pathologist to be familiar with the FISH probes, reagents, diagnostic panels, and methods used to produce the stained preparation. Staining must be performed in a certified, licensed laboratory under the supervision of a pathologist who is responsible for reviewing the stained slides and assuring the adequacy of positive and negative controls.
- Specimen staining, especially signal intensity and background staining, is dependent on the handling and processing of the specimen prior to staining. Improper fixation, freezing, thawing, washing, drying, heating, sectioning, or contamination with other specimens or fluids may produce artefacts or false results. Inconsistent results may result from variations in fixation and embedding methods, as well as from inherent irregularities within the specimen.
- The probe should be used only for detecting loci described in 4. "Reagents provided".
- The performance was validated using the procedures described in this instruction for use. Modifications to these procedures might alter the performance and have to be validated by the user.

## 9. Interfering substances

Red blood cells present in the specimen might exhibit autofluorescence which hinders signal recognition.

The following fixatives are incompatible with FISH:

- Bouin's fixative
- B5 fixative
- Acidic fixatives (e.g., picric acid)
- Zenker's fixative
- Alcohols (when used alone)
- Mercuric chloride
- Formaldehyde/zinc fixative
- Hollande's fixative
- Non-buffered formalin

# 10. Preparation of specimens

# Cytology Specimens

 Prepare specimens as described in the instructions for use of the <u>ZytoLight FISH-Cytology Implementation Kit</u>.

# **FFPE Specimens**

- Fixation in 10% neutrally buffered formalin for 24 h at room temperature (18-25°C).
- Sample size  $\leq 0.5$  cm<sup>3</sup>.
- Use premium quality paraffin.
- Embedding should be carried out at temperatures lower than 65°C.
- Prepare 2-4 μm microtome sections.
- Use positively charged microscope slides.
- Fix for 2-16 h at 50-60°C.

# 11. Preparatory treatment of the device

The product is ready-to-use. No reconstitution, mixing, or dilution is required. Bring probe to room temperature (18-25°C) before use, protect from light. Prior to opening the vial, mix by vortexing and spin down briefly.

## 12. Assay procedure

## Cytology Specimens

# Specimen pretreatment

Perform specimen pretreatment according to the instructions for use of the Zyto Light FISH-Cytology Implementation Kit.

# Denaturation and hybridization

- 1. Pipette 10  $\mu$ l of the probe onto each pretreated specimen.
- 2. Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

- Place slides on a hot plate or hybridizer and denature specimens for 5 min at 72°C.
- **4.** Perform hybridization for 2 h up to 16 h (i.e., overnight) at 37°C by either transferring the slides to a hybridizer or to a humidity chamber and a hybridization oven.

It is essential that specimens do not dry out during the hybridization step.

# Post-hybridization

Perform post-hybridization processing (washing, counter-staining, fluorescence microscopy) according to the instructions for use of the <u>ZytoLight FISH-Cytology Implementation Kit</u>.

# **FFPE Specimens**

## Specimen pretreatment

Perform specimen pretreatment (dewaxing, proteolysis) according to the instructions for use of the Zyto Light FISH-Tissue Implementation Kit.

# Denaturation and hybridization

- 1. Pipette  $10 \,\mu\text{l}$  of the probe onto each pretreated specimen.
- 2. Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

- Place slides on a hot plate or hybridizer and denature specimens for 10 min at 75°C
- Transfer slides to a humidity chamber and hybridize overnight at 37°C (e.g., in a hybridization oven).

It is essential that specimens do not dry out during the hybridization step.

#### Post-hybridization

Perform post-hybridization processing (washing, counter-staining, fluorescence microscopy) according to the instructions for use of the ZytoLight FISH-Tissue Implementation Kit

## 13. Interpretation of results

With the use of appropriate filter sets, the hybridization signals of the probe appear green (distal to the MYC breakpoint region) and orange (proximal to the MYC breakpoint region).

**Normal situation**: In interphases of normal cells or cells without a translocation involving the MYC gene region, two green/orange fusion signals appear (see Fig. 2).

**Aberrant situation:** One MYC gene region affected by a translocation is indicated by one separate green signal and one separate orange signal (see Fig. 2).

Overlapping signals may appear as yellow signals.

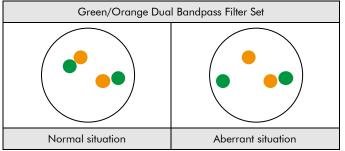


Fig. 2: Expected results in normal and aberrant nuclei

Alternative breakpoints, particularly observed in variant MYC translocations t(8;22) and t(2;8), might result in different signal patterns than described above or in false negative signal patterns. Please check localization of the probe (see Fig. 1) carefully. Unexpected signal patterns or results should be further investigated.

# Please note:

- Due to decondensed chromatin, single FISH signals can appear as small signal clusters. Thus, two or three signals of the same size, separated by a distance ≤ 1 signal diameter, should be counted as one signal.
- Do not evaluate overlapping nuclei.
- Do not count over-digested nuclei (recognized by dark areas visible inside of the nuclei).
- Do not count nuclei with strong auto-fluorescence, which hinders signal recognition.
- A negative or unspecific result can be caused by multiple factors (see chapter 17).

# 14. Recommended quality control procedures

In order to monitor correct performance of processed specimens and test reagents, each assay should be accompanied by internal and external controls. If internal and/or external controls fail to demonstrate appropriate staining, results with patient specimens must be considered invalid.

**Internal control:** Non-neoplastic cells within the specimen that exhibit normal signal pattern.

External control: Validated positive and negative control specimens.

#### 15. Performance characteristics

# Cytology Specimens

The performance was evaluated according to the instructions for use of the Zyto Light FISH-Cytology Implementation Kit.

**Accuracy:** The location of hybridization of the probe was evaluated on metaphase spreads of a karyotypically normal male. In all tested specimens the probe hybridized solely to the expected loci. No additional signals or cross-hybridizations were observed. Therefore, the accuracy was calculated to be 100%.

**Analytical sensitivity:** For the analytical sensitivity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. All nuclei showed the expected normal signal pattern in all tested specimens. Therefore, the analytical sensitivity was calculated to be 100%.

**Analytical specificity:** For the analytical specificity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. In all tested specimens, all signals hybridized solely to the expected target loci and no other loci. Therefore, the analytical specificity was calculated to be 100%.

# **FFPE Specimens**

The performance was evaluated according to the instructions for use of the Zyto *Light* FISH-Tissue Implementation Kit.

**Accuracy:** The location of hybridization of the probe was evaluated on metaphase spreads of a karyotypically normal male. In all tested specimens the probe hybridized solely to the expected loci. No additional signals or cross-hybridizations were observed. Therefore, the accuracy was calculated to be 100%.

**Analytical sensitivity:** For the analytical sensitivity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. All nuclei showed the expected normal signal pattern in all tested specimens. Therefore, the analytical sensitivity was calculated to be 100%.

**Analytical specificity:** For the analytical specificity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. In all tested specimens, all signals hybridized solely to the expected target loci and no other loci. Therefore, the analytical specificity was calculated to be 100%.

# 16. Disposal

The disposal of reagents must be carried out in accordance with local regulations.

#### 17. Troubleshooting

Any deviation from the operating instructions can lead to inferior staining results or to no staining at all. Some of the tips in this section only apply when using the Zyto Light FISH-Tissue Implementation Kit.

Weak signals or no signals at all

Possible cause	Action
No target sequences available	Use appropriate controls
Cell or tissue sample not fixed properly	Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the Zyto Light FISH-Tissue Implementation Kit
Heat pretreatment, proteolysis, denaturation, hybridization, or stringency wash temperature incorrect	Check temperature of all technical devices used, using a calibrated thermometer
Proteolytic pretreatment not carried out properly	Optimize pepsin incubation time, increase or decrease if necessary

Probe evaporation	When using a hybridizer, the use of the wet stripes/water filled tanks is mandatory. When using a hybridization oven, the use of a humidity chamber is required. In addition, the coverslip should be sealed completely, e.g., with Fixogum, to prevent drying-out of the sample during hybridization
Too low concentrated stringency wash buffer	Check concentration of stringency wash buffer
Old dehydration solutions	Prepare fresh dehydration solutions
Fluorescence microscope adjusted wrongly	Adjust correctly
Inappropriate filter sets used	Use filter sets appropriate for the fluochromes of the probe.  Triple-bandpass filter sets provide less light compared to single or dualbandpass filter sets. Consequently, the signals may appear fainter using these triple-bandpass filter sets
Photo-damage of the probes/fluorophores	Accomplish hybridization and washing steps in the dark

Cross hybridization signals; noisy background

Possible cause	Action
Incomplete dewaxing	Use fresh solutions; check duration of dewaxing
Proteolytic pretreatment too strong	Reduce pepsin incubation time
Probe volume per area too high	Reduce probe volume per section/area, distribute probe dropwise to avoid local concentration
Slides cooled to room temperature before hybridization	Transfer the slides quickly to 37°C
Too high concentrated stringency wash buffer	Check concentration of stringency wash buffer
Washing temperature following hybridization too low	Check temperature; increase if necessary
Dehydration of specimens between the individual incubation steps	Prevent dehydration by sealing the slides and performing incubation in a humid environment

Morphology degraded

Morphology degraded		
Possible cause	Action	
Cell or tissue sample has not been fixed properly	Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the ZytoLight FISH-Tissue Implementation Kit	
Proteolytic pretreatment not carried out properly	Optimize pepsin incubation time, increase or decrease if necessary	
Insufficient drying before probe application	Extend air-drying	

Overlappina nuclei

- 1 - 1 - 1 - 1 - 1 - 1 - 1 - 1 - 1 - 1		
Possible cause	Action	
Inappropriate thickness of tissue sections	Prepare 2-4 $\mu$ m microtome sections	

## Specimen floats off the slide

Possible cause	Action
Unsuitable slide coating	Use appropriate slides
Proteolytic pretreatment too strong	Reduce pepsin incubation time

#### Weak counterstain

Possible cause	Action
Low concentrated DAPI solution	Use <u>DAPI/DuraTect-Solution (ultra)</u> (Prod. No. MT-0008-0.8) instead
DAPI incubation time too short	Adjust DAPI incubation time

## 18. Literature

- Haralambieva E, et al. (2004) Genes Chromosomes Cancer 40: 10-8
- Kievits T, et al. (1990) Cytogenet Cell Genet 53: 134-6.
- Veronese ML, et al. (1995) Blood 85: 2132-8.
- Wilkinson DG: In Situ Hybridization, A Practical Approach, Oxford University Press (1992) ISBN 0 19 963327 4.

Our experts are available to answer your questions. Please contact  $\frac{\text{helptech}@zytovision.com}{}$ 



ZytoVision GmbH Fischkai 1

27572 Bremerhaven/ Germany Phone: +49 471 4832-300 Fax: +49 471 4832-509 www.zytovision.com

Email: info@zytovision.com

### Trademarks:

 ${\it ZytoVision}^{\it @} \ {\it and} \ {\it ZytoLight}^{\it @} \ {\it are} \ {\it trademarks} \ {\it of} \ {\it ZytoVision} \ {\it GmbH}.$