

Zyto Light SPEC TP53/CEN 17 Dual Color Probe

REF Z-2153-50

 \sum

5 (0.05 ml)

REF Z-2153-200



20 (0.2 ml)

For the qualitative detection of human TP53 gene deletions and alpha satellites of chromosome 17 by fluorescence *in situ* hybridization (FISH)



In vitro diagnostic medical device according to EU directive 98/79/EC

1. Intended use

The <u>ZytoLight SPEC TP53/CEN 17 Dual Color Probe</u> (**PL109**) is intended to be used for the qualitative detection of human TP53 gene deletions as well as the detection of chromosome 17 alpha satellites in cytologic or formalin-fixed, paraffin-embedded specimens such as leukemic cells by fluorescence *in situ* hybridization (FISH). The probe is intended to be used in combination with <u>ZytoLight FISH Implementation Kits</u> (Prod. No. Z-2028-5/-20, or Z-2099-20).

Interpretation of the results must be made within the context of the patient's clinical history with respect to further clinical and pathologic data of the patient by a qualified pathologist.

2. Clinical relevance

The TP53 gene (tumor protein 53) is located in the chromosomal region 17p13.1 and encodes a 53 kDa transcription factor which regulates cell proliferation, differentiation, and apoptosis and which functions as a tumor suppressor by activating the expression of genes that inhibit cell growth. Deletions affecting the short arm of chromosome 17 (17p), the site of the TP53 gene, are often accompanied by mutations in the remaining allele, and thus result in the loss of TP53 tumor suppressor activity. TP53 gene deletions have been detected in patients with chronic lymphocytic leukemia (CLL), multiple myeloma (MM), acute myeloid leukemia (AML), and are also very frequent in primary solid tumors of different histological origin. The presence of TP53 deletion has been shown to correlate with more aggressive disease, shortened survival, and poor response to standard treatment. CLL patients with deletion of 17p are more likely to respond to treatment with the monoclonal anti-CD52 antibody alemtuzumab than to conventional chemotherapy. FISH is an effective method to screen for deletions affecting the TP53 gene locus in order to identify patients who are candidates for alternative treatment and to avoid administration of otherwise ineffective therapy.

3. Test principle

The fluorescence *in situ* hybridization (FISH) technique allows for the detection and visualization of specific nucleic acid sequences in cell preparations. Fluorescently-labeled DNA fragments, so called FISH probes, and their complementary target DNA strands in the preparations are co-denatured and subsequently allowed to anneal during hybridization. Afterwards, unspecific and unbound probe fragments are removed by stringency washing steps. After counterstaining the DNA with DAPI, hybridized probe fragments are visualized using a fluorescence microscope equipped with excitation and emission filters specific for the fluorochromes with which the FISH probe fragments have been directly labeled.

4. Reagents provided

The ZytoLight SPEC TP53/CEN 17 Dual Color Probe is composed of:

- ZyOrange (excitation 547 nm/emission 572 nm) labeled polynucleotides (~4.5 ng/μl), which target sequences mapping in 17p13.1* (chr17:7,495,749-7,663,022) harboring the TP53 gene region (see Fig. 1).
- ZyGreen (excitation 503 nm/emission 528 nm) labeled polynucleotides (~4.5 ng/μl), which target sequences mapping in 17p11.1-q11.1 specific for the alpha satellite centromeric region D17Z1 of chromosome 17.
- Formamide based hybridization buffer

*according to Human Genome Assembly GRCh37/hg19

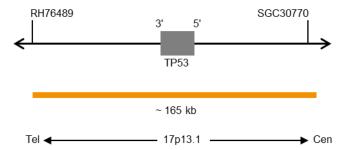


Fig. 1: SPEC TP53 Probe map (not to scale)

The Zyto*Light* SPEC TP53/CEN 17 Dual Color Probe is available in two sizes:

- Z-2153-50: 0.05 ml (5 reactions of 10 μl each)
- Z-2153-200: 0.2 ml (20 reactions of 10 μl each)

5. Materials required but not provided

- Positive and negative control specimens
- Hybridizer or hot plate
- Hybridizer or humidity chamber in hybridization oven
- Timer
- Staining jars or baths
- Calibrated thermometer
- Adjustable pipettes (10 μl, 25 μl)
- Ethanol or reagent alcohol
- Deionized or distilled water
- Coverslips (22 mm x 22 mm, 24 mm x 60 mm)
- Rubber cement, e.g., <u>Fixogum Rubber Cement</u> (Prod. No. E-4005-50/-125) or similar
- Adequately maintained fluorescence microscope (400-1000x)
- Immersion oil approved for fluorescence microscopy
- Appropriate filter sets

Cytology Specimens

- <u>Zyto Light</u> FISH-Cytology Implementation Kit (Prod. No. Z-2099-20)
- Microscope slides, uncoated
- Water bath (70°C)
- 37% formaldehyde, acid-free, or 10% formalin, neutrally buffered
- 2x Saline-Sodium Citrate (SSC), e.g., from 20x SSC Solution (Prod. No. WB-0003-50)

1/4 2019-03-13

FFPE Specimens

- <u>ZytoLight</u> FISH-Tissue Implementation Kit (Prod. No. Z-2028-5/-20)
- Microscope slides, positively charged
- Water bath (37°C, 98°C)
- Xvlene

6. Storage and handling

Store at 2-8°C in an upright position protected from light.

Use protected from light. Return to storage conditions immediately after use. Do not use reagents beyond expiry date indicated on the label. The product is stable until expiry date indicated on the label when handled accordingly.

7. Warnings and precautions

- Read the instruction for use prior to use!
- Do not use the reagents after the expiry date has been reached!
- This product contains substances (in low concentrations and volumes) that are harmful to health and potentially infectious. Avoid any direct contact with the reagents. Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- If reagents come into contact with skin, rinse skin immediately with copious quantities of water!
- A material safety data sheet is available on request for the professional user.
- Do not reuse reagents.
- Avoid cross-contamination of samples as this may lead to erroneous results.
- The probe should not be exposed to light, especially strong light, for a longer period of time, i.e., all steps should be accomplished, where possible, in the dark and/or using lightproof containers!

Hazard and precautionary statements:

The hazard determining component is Formamide.



Danger

H351	Suspected of causing	cancer.
HOLOED	AA	A AI -

H360FD May damage fertility. May damage the unborn child.
H373 May cause damage to organs through prolonged or

repeated exposure.

P201 Obtain special instructions before use.

P202 Do not handle until all safety precautions have been read

and understood.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.
P280 Wear protective gloves/protective clothing/eye

protection/face protection.

P308+P313 IF exposed or concerned: Get medical advice/attention.

P405 Store locked up.

8. Limitations

- For in vitro diagnostic use.
- For professional use only.
- The clinical interpretation of any positive staining, or its absence, must be done within the context of clinical history, morphology, other histopathological criteria as well as other diagnostic tests. It is the responsibility of a qualified pathologist to be familiar with the FISH probes, reagents, diagnostic panels, and methods used to produce the stained preparation. Staining must be performed in a certified, licensed laboratory under the supervision of a pathologist who is responsible for reviewing the stained slides and assuring the adequacy of positive and negative controls.
- Specimen staining, especially signal intensity and background staining, is dependent on the handling and processing of the specimen prior to staining. Improper fixation, freezing, thawing, washing, drying,

heating, sectioning, or contamination with other specimens or fluids may produce artefacts or false results. Inconsistent results may result from variations in fixation and embedding methods, as well as from inherent irregularities within the specimen.

- The probe should be used only for detecting loci described in 4. "Reagents provided".
- The performance was validated using the procedures described in this instruction for use. Modifications to these procedures might alter the performance and have to be validated by the user.

9. Interfering substances

Red blood cells present in the specimen might exhibit autofluorescence which hinders signal recognition.

The following fixatives are incompatible with FISH:

- Bouin's fixative
- B5 fixative
- Acidic fixatives (e.g., picric acid)
- Zenker's fixative
- Alcohols (when used alone)
- Mercuric chloride
- Formaldehyde/zinc fixative
- Hollande's fixative
- Non-buffered formalin

10. Preparation of specimens

Cytology Specimens

 Prepare specimens as described in the instructions for use of the <u>ZytoLight</u> FISH-Cytology Implementation Kit.

FFPE Specimens

- Fixation in 10% neutrally buffered formalin for 24 h at room temperature (18-25°C).
- Sample size ≤ 0.5 cm³.
- Use premium quality paraffin.
- Embedding should be carried out at temperatures lower than 65°C.
- Prepare 2-4 μm microtome sections.
- Use positively charged microscope slides.
- Fix for 2-16 h at 50-60°C.

11. Preparatory treatment of the device

The product is ready-to-use. No reconstitution, mixing, or dilution is required. Bring probe to room temperature ($18-25^{\circ}$ C) before use, protect from light. Prior to opening the vial, mix by vortexing and spin down briefly.

12. Assay procedure

Cytology Specimens

Specimen pretreatment

Perform specimen pretreatment according to the instructions for use of the Zyto *Light* FISH-Cytology Implementation Kit.

Denaturation and hybridization

- 1. Pipette 10 μ l of the probe onto each pretreated specimen.
- 2. Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

- **3.** Place slides on a hot plate or hybridizer and denature specimens for 5 min at 72°C.
- 4. Transfer the slide to a humidity chamber and hybridize overnight at 37°C (e.g., in a hybridization oven).

It is essential that specimens do not dry out during the hybridization step.

Post-hybridization

Perform post-hybridization processing (washing, counter-staining, fluorescence microscopy) according to the instructions for use of the ZytoLight FISH-Cytology Implementation Kit.

2/4 2019-03-13

FFPE Specimens

Specimen pretreatment

Perform specimen pretreatment (dewaxing, proteolysis) according to the instructions for use of the <u>ZytoLight FISH-Tissue Implementation Kit</u>.

Denaturation and hybridization

- 1. Pipette $10 \mu l$ of the probe onto each pretreated specimen.
- Cover specimens with a 22 mm x 22 mm coverslip (avoid trapped bubbles) and seal the coverslip.

We recommend using rubber cement (e.g., Fixogum) for sealing.

- Place slides on a hot plate or hybridizer and denature specimens for 10 min at 75°C
- **4.** Transfer slides to a humidity chamber and hybridize overnight at 37°C (e.g., in a hybridization oven).

It is essential that specimens do not dry out during the hybridization step.

Post-hybridization

Perform post-hybridization processing (washing, counter-staining, fluorescence microscopy) according to the instructions for use of the ZytoLight FISH-Tissue Implementation Kit

13. Interpretation of results

With the use of appropriate filter sets, the hybridization signals of the probe appear orange (TP53 gene region) and green (CEN 17).

Normal situation: In interphases of normal cells or cells without a deletion involving the TP53 gene region, two orange signals and two green signals appear (see Fig. 2).

Aberrant situation: In a cell with deletion affecting the TP53 gene region, a reduced number of orange signals will be observed. Deletions affecting only parts of the TP53 gene region might result in a normal signal pattern with orange signals of reduced size (see Fig. 2).

Overlapping signals may appear as yellow signals.

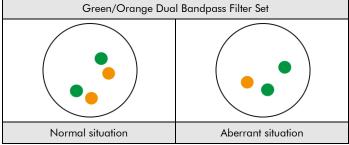


Fig. 2: Expected results in normal and aberrant nuclei

Other signal distribution may be observed in some abnormal samples which might result in a different signal pattern than described above, indicating variant rearrangements. Unexpected signal patterns should be further investigated.

Please note:

- Due to decondensed chromatin, single FISH signals can appear as small signal clusters. Thus, two or three signals of the same size, separated by a distance ≤ 1 signal diameter, should be counted as one signal.
- Do not evaluate overlapping nuclei.
- Do not count over-digested nuclei (recognized by dark areas visible inside of the nuclei).
- Do not count nuclei with strong auto-fluorescence, which hinders signal recognition.
- A negative or unspecific result can be caused by multiple factors (see chapter 17).
- In order to correctly interpret the results, the user must validate this
 product prior to use in diagnostic procedures according to national
 and/or international guidelines.

14. Recommended quality control procedures

In order to monitor correct performance of processed specimens and test reagents, each assay should be accompanied by internal and external controls. If internal and/or external controls fail to demonstrate appropriate staining, results with patient specimens must be considered invalid.

Internal control: Non-neoplastic cells within the specimen that exhibit normal signal pattern.

External control: Validated positive and negative control specimens.

15. Performance characteristics

Cytology Specimens

The performance was evaluated according to the instructions for use of the Zyto *Light* FISH-Cytology Implementation Kit.

Accuracy: The location of hybridization of the probe was evaluated on metaphase spreads of a karyotypically normal male. In all tested specimens the probe hybridized solely to the expected loci. No additional signals or cross-hybridizations were observed. Therefore, the accuracy was calculated to be 100%.

Analytical sensitivity: For the analytical sensitivity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. All nuclei showed the expected normal signal pattern in all tested specimens. Therefore, the analytical sensitivity was calculated to be 100%.

Analytical specificity: For the analytical specificity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. In all tested specimens, all signals hybridized solely to the expected target loci and no other loci. Therefore, the analytical specificity was calculated to be 100%.

FFPE Specimens

The performance was evaluated according to the instructions for use of the Zyto Light FISH-Tissue Implementation Kit.

Accuracy: The location of hybridization of the probe was evaluated on metaphase spreads of a karyotypically normal male. In all tested specimens the probe hybridized solely to the expected loci. No additional signals or cross-hybridizations were observed. Therefore, the accuracy was calculated to be 100%.

Analytical sensitivity: For the analytical sensitivity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. All nuclei showed the expected normal signal pattern in all tested specimens. Therefore, the analytical sensitivity was calculated to be 100%.

Analytical specificity: For the analytical specificity assessment, the probe was evaluated on metaphase spreads of karyotypically normal males. In all tested specimens, all signals hybridized solely to the expected target loci and no other loci. Therefore, the analytical specificity was calculated to be 100%.

16. Disposal

The disposal of reagents must be carried out in accordance with local regulations.

17. Troubleshooting

Any deviation from the operating instructions can lead to inferior staining results or to no staining at all. Some of the tips in this section only apply when using the Zyto Light FISH-Tissue Implementation Kit.

3/4 2019-03-13

Weak signals or no signals at all

Possible cause	Action
No target sequences available	Use appropriate controls
Cell or tissue sample not fixed properly	Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the <u>Zyto Light</u> FISH-Tissue <u>Implementation Kit</u>
Heat pretreatment, proteolysis, denaturation, hybridization, or stringency wash temperature incorrect	Check temperature of all technical devices used, using a calibrated thermometer
Proteolytic pretreatment not carried out properly	Optimize pepsin incubation time, increase or decrease if necessary
Probe evaporation	When using a hybridizer, the use of the wet stripes/water filled tanks is mandatory. When using a hybridization oven, the use of a humidity chamber is required. In addition, the coverslip should be sealed completely, e.g., with Fixogum, to prevent drying-out of the sample during hybridization
Too low concentrated stringency wash buffer	Check concentration of stringency wash buffer
Old dehydration solutions	Prepare fresh dehydration solutions
Fluorescence microscope adjusted wrongly	Adjust correctly
Inappropriate filter sets used	Use filter sets appropriate for the fluochromes of the probe. Triple-bandpass filter sets provide less light compared to single or dualbandpass filter sets. Consequently, the signals may appear fainter using these triple-bandpass filter sets
Photo-damage of the probes/fluorophores	Accomplish hybridization and washing steps in the dark

Cross hybridization signals; noisy background

Cross hybridization signals; noisy background		
Possible cause	Action	
Incomplete dewaxing	Use fresh solutions; check duration of dewaxing	
Proteolytic pretreatment too strong	Reduce pepsin incubation time	
Probe volume per area too high	Reduce probe volume per section/area, distribute probe dropwise to avoid local concentration	
Slides cooled to room temperature before hybridization	Transfer the slides quickly to 37°C	
Too high concentrated stringency wash buffer	Check concentration of stringency wash buffer	
Washing temperature following hybridization too low	Check temperature; increase if necessary	
Dehydration of specimens between the individual incubation steps	Prevent dehydration by sealing the slides and performing incubation in a humid environment	

Morphology degraded	
Possible cause	Action
Cell or tissue sample has not been fixed properly	Optimize fixing time and fixative or apply a post-fixation step as described in "assay procedure" of the manual of the <u>ZytoLight</u> <u>FISH-Tissue Implementation Kit</u>
Proteolytic pretreatment not carried out properly	Optimize pepsin incubation time, increase or decrease if necessary
Insufficient drying before probe application	Extend air-drying

Overlapping nuclei

Possible cause	Action
Inappropriate thickness of tissue sections	Prepare 2-4 μm microtome sections

Specimen floats off the slide

Specimen nodis on me side	
Possible cause	Action
Unsuitable slide coating	Use appropriate slides
Proteolytic pretreatment too strong	Reduce pepsin incubation time

Weak counterstain

Possible cause	Action
Low concentrated DAPI solution	Use <u>DAPI/DuraTect-Solution (ultra)</u> (Prod. No. MT-0008-0.8) instead
DAPI incubation time too short	Adjust DAPI incubation time

18. Literature

- Amiel A, et al. (1997) Cancer Genet Cytogenet 97: 97-100.
- Chang H, et al. (2005) *Blood* 105: 358-60.
- Chang H, et al. (2010) Am J Clin Pathol 133: 70-4.
- Herrera JC, et al. (2010) Biomedica 30: 390-400.
- Kievits T, et al. (1990) Cytogenet Cell Genet 53: 134-6.
- Lozanski G, et al. (2004) Blood 103: 3278-81.
- Tavor S, et al. (2011) Leuk Lymphoma 52: 642-7.
- Wilkinson DG: In Situ Hybridization, A Practical Approach, Oxford University Press (1992) ISBN 0 19 963327 4.

Our experts are available to answer your questions. Please contact <u>helptech@zytovision.com</u>



ZytoVision GmbH Fischkai 1

27572 Bremerhaven/ Germany Phone: +49 471 4832-300 Fax: +49 471 4832-509 www.zytovision.com

Email: info@zytovision.com

Trademarks:

ZytoVision® and ZytoLight® are trademarks of ZytoVision GmbH.

2019-03-13 4/4